

# CVS2

## CVS2 - Online Computerized Vergence System

---

### Doctor's Manual

© Copyright HTS Inc. 2019



**CVS2 - Online Computerized Vergence System**

**Developed By**

Dr. Jeffery Cooper, OD, FCOVD

Rodney K. Bortel

Ted L. Conner

# TABLE OF CONTENTS

Registering CVS2	2
Running CVS2	3
Dispensing CVS2 Kits	4
Active Patients List	6
Managing Existing Patients	8
Modify Existing Patients Procedures	10
Performance Overview	13
Procedure Performance	16
Contacting Your Patients	19
Customize New Patient's Program	24
Modify New Patient's Procedures	25
Contacting HTS	28
Demonstrating CVS2	29
Divergence	31
Convergence	41
Jump Ductions	51
Jump Random	61
Vergence Base Up	71
Vergence Base Down	76

## REGISTERING CVS2

Register your CVS2 Program by following the directions on the CVS2 Registration Card, included with your CVS2 Kit.

### CVS2 Registration Card

#### Device Compatibility

CVS2 Online Computerized Vergence System is designed to work on Chrome, Safari, or Silk Browsers.

Internet Explorer and Microsoft Edge are not supported but Google Chrome is a free alternative.

Please call HTS or email us if you wish to install Google Chrome and need assistance.

CVS2 Online Computerized Vergence System will work with most Desktops, Laptops, and Tablets(Including iPads, Androids, Chromebooks).

CVS2 will not work on cell phones or devices with a screen smaller than 7.5 inches in width.

#### Instructions for registering your CVS2

To get started, open your browser and go to: [HTS.Vision](https://HTS.Vision)

At the Home Page, Click/Touch Continue.

On the Welcome Page, Click/Touch the Register button.

After reading the End User License Agreement, Click/Touch on Accept at the bottom of the page to proceed.

You will be prompted for a registration code which is:

After entering the registration code, you will be prompted to create a permanent Username and Password for login.

The Username and Password you create will be used to login in the future.


Be sure to make a record of your Username and Passcode on the reverse side of this card before clicking Continue.



Tel: 800-346-4925  
Intl Tel: 480-983-0857  
Email: [CVS2@homevisiontherapy.com](mailto:CVS2@homevisiontherapy.com)

## RUNNING CVS2

After your first login, you will see the Active Patients List.

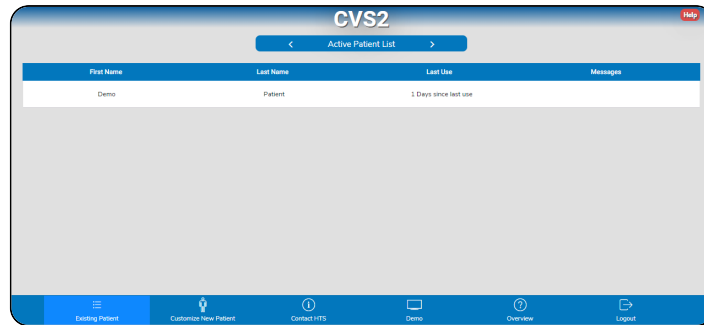

This lists the patients that are currently registered to your account.

Initially, the Demo Patient will be the only patient available.

The Navigation Bar at the bottom of the screen allows you to: Modify the program for Existing Patients and view their progress, Customize a New Patient's program before dispensing, Contact HTS, Demonstrate/Practice the procedures in CVS2, and review the CVS2 operating manual from the Overview button.

You may Click the red **HELP** button in the upper right of any page to view detailed information about that page.

## Dispensing CVS2 to a new Patient

An CVS2 Patient's Kit will include the following materials: CVS2 Overview Guide, which provides an overview for the patient of how CVS2 works, The CVS2 Registration Card with instructions for activating the CVS2 on the patient's devices and 1 pair of Red/Blue Glasses.

The CVS2 Patients program comes loaded with 150 runs. If your patient uses all 150 and you feel they need more exercise, HTS will add more runs at your direction.

CVS2 program is configured with the default Daily Exercise Regimen "Today's Assignment" which is designed to address most Convergence insufficiency cases.

The default Daily Exercise Regimen includes the following procedures:

<b>Divergence</b>	Seven minutes every day until > 13 Prism Diopters has been achieved on the larger target three times. Divergence is complete after > 13 Prism Diopters has been achieved on the small target six times and all stars have been earned.
<b>Convergence</b>	Seven minutes every day until > 35 Prism Diopters has been achieved on the larger target three times. Convergence is complete after > 35 Prism Diopters has been achieved on the small target six times and all stars have been earned.
<b>Jump Ductions</b>	Requires Convergence and Divergence to be complete. Seven minutes every day until > 13 Prism Diopters Divergence and > 35 Prism Diopters Convergence have been achieved on the larger target seven times. Jump Ductions is complete after > 13 Prism Diopters Divergence and > 35 Prism Diopters Convergence have been achieved on the small target eight times and all stars have been earned.
<b>Jump Random</b>	Requires Jump Ductions to be complete. Seven minutes every day until > 80 Percent Correct Convergence and Divergence have been achieved on the larger target seven times. Jump Random is complete after > 80 Percent Correct Convergence and Divergence have been achieved on the small target eight times and all stars have been earned.

Each Exercise has preset goals. When the patient achieves the goal for a procedure during a therapy session, they will earn a "Star" for that procedure. When all the "Stars" for a procedure have been earned, it will drop out of the Today's Assignment.

When Divergence and Convergence have been mastered, they each will drop out of Today's Assignment and be replaced by Jump Ductions. When Jump Ductions has been mastered, it will drop out and be replaced by Jump Ductions Random.

When all the procedures have been mastered, you will receive a message that your patient has mastered the default Daily Exercise Regimen.

At that time they will automatically continue with Jump Random until their runs expire or, if you wish, you may instruct the patient to terminate therapy.

The default Daily Exercise Regimen may be modified either before or after dispensing by adding or removing procedures and changing settings and goals depending upon the severity of the case. The exercises you choose to include will be automatically presented in the Daily Exercise Regimen. They will continue until their assigned goals have been achieved. If you wish an exercise to be assigned with no endpoint but still be presented in the Daily Exercise Regimen, select Customize New Patient, choose that exercise, click Edit Settings then select Maintenance.



To have your patient use the CVS2 with the default Daily Exercise Regimen, simply dispense the CVS2 Kit as provided by CVS. They will follow the steps provided on the Registration Card to get started. They will be provided with operating instructions while using the CVS2 program and will see tutorials regarding each procedure.

## Active Patients List Page



On this page, if you wish, view your patients and their status at a glance. You may view messages from your patient, their scores, and other details by Clicking a patients name in the list.

### To View Your Patients


When a patient has registered their CVS2, they will automatically appear in the active patient list.

First Name	Last Name	Last Use	Messages
Demo	Patient	1 Days since Last Use	
James	Brown	5 Days since Last Use	
Sophia	Garcia	1 Days since Last Use	
Emma	Smith	2 Days since Last Use	
Ben	Wilson	1 Days since Last Use	

You may Click the First Name or Last Name column to sort from A-Z or Z-A.

First Name	Last Name	Last Use	Messages
Demo	Patient	1 Days since Last Use	
Ben	Wilson	1 Days since Last Use	
Emma	Smith	2 Days since Last Use	
James	Brown	5 Days since Last Use	
Sophia	Garcia	1 Days since Last Use	

Days Since Last Use displays the last time the patient used the program.

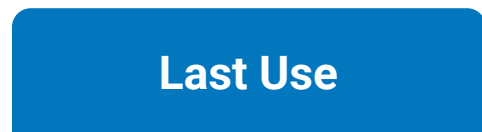

1 Days Since Last Use

An unread message from your patient is denoted by the mail icon. If you have multiple unread messages, a number will be displayed on the icon.



## Messages



You may Click the "Active Patients List" button at the top of the screen to switch between your Active and Inactive Patients.




To manage existing patients, Click on either their First or Last name.

First Name	Last Name	Last Use	Messages
Demo	Patient	1 Days since Last Use	
Ben	Wilson	5 Days since Last Use	
Emma	Smith	1 Days since Last Use	
James	Brown	2 Days since Last Use	
Sophia	Garcia	1 Days since Last Use	



## Managing your Existing Patients

If you wish to modify a patient's program or view their scores/progress, Click on their name from the Active Patients list, to display the Patient Management Page.

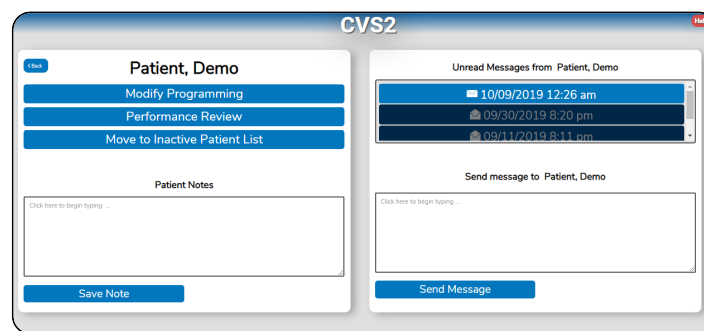

Click the red **HELP** button at the top of any of these page for detailed information.

On this page, you may make Programming modifications to the default Daily Therapy Protocol. Patient's scores may be accessed from the Performance Review button. Move patients from the Active to the Inactive Patient list. You may exchange messages with your patient and record data for the patient's record.

## To Manage Your Patients

To Modify Procedure Parameters, click the Modify Programming button.

Modify Programming

To View Patient Performance data and progress, click the Performance Review button.

Performance Review

To archive an active patient, click Move to Inactive Patient List. Conversely, you may unarchive a patient by touching the Move to Active Patient List button.

Move to Active Patient List

## Modify Existing Patient's Procedures & Protocol

If the CVS2 program has already been dispensed and you wish to modify their programming, you may do so on this page.

### To modify the Daily Exercise Regimen

A procedure and its settings are displayed as a white card.

The screenshot shows a white card for a procedure named 'Convergence'. At the top left is a blue checkmark icon. To its right is the text 'Convergence' and 'Assigned'. Further right is a red link 'About Convergence'. Below this, on the right side, is the text 'Procedure Completed Maintenance' in green. The main body of the card contains a table of settings: 'Diopters: 35', 'Duration: 7', and 'Response Timeout: 10'. On the right side of the card, there are two blue buttons: 'Reset Stars' with a star icon and 'Edit Settings' with a pencil icon.

For more details on a procedure, click on **About ...** link to see the operating instructions for the procedure.

### About Convergence

A blue checkbox next to a procedure's name denotes if it is assigned to the Daily Exercise Regimen. Click the checkbox to add or remove a procedure from the Protocol.

This screenshot is identical to the one above, but it includes a mouse cursor pointing at the blue checkmark icon in the top left corner of the procedure card.


To edit any procedure's settings, click Edit Settings.



Click the - or + buttons to change criteria and goal settings.

Click the checkbox to enable or disable additional features.

Click the ? to see additional information about features and settings.

Close X	Convergence	Done ✓
<input type="button" value="-"/> 35 <input type="button" value="+"/>	Diopters	<input type="button" value="?"/>
<input type="button" value="-"/> 8 <input type="button" value="+"/>	Duration	<input type="button" value="?"/>
<input type="button" value="-"/> 10 <input type="button" value="+"/>	Response Timeout	<input type="button" value="?"/> 
	<input type="checkbox"/> Maintenance	<input type="button" value="?"/>
	<input type="checkbox"/> Auditory Distractors	<input type="button" value="?"/>

Click Done ✓ when satisfied with your selections.




Then click Save Changes.



## While editing settings

A procedure that has all its Stars earned and is no longer assigned to the Daily Exercise Regimen will display Procedure Complete.

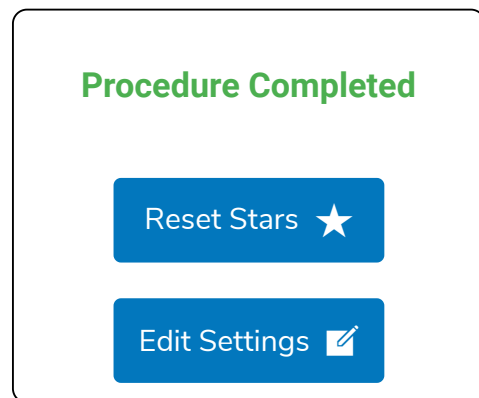

You may reset the Stars for a procedure in order to reinsert that procedure in the Daily Exercise Regimen if all the stars have already been earned. Selecting Reset Stars will erase all the earned stars and reassign that procedure to the Daily Exercise Regimen until all the stars have again been earned.



If you wish to have your patient continue a procedure after earning all the Stars, select Maintenance On. The procedure then will present in the Daily Assignment until you disable the procedure by clicking the blue checkbox that it reads Maintenance Off.

Maintenance Mode On

## Performance Overview Page

This page displays the patient's progress and compliance at a glance. Click any procedure name for more detailed results of that procedure.


### Using the Calendar

The Calendar may be used to quickly assess your patient's compliance. Dates that the entire daily assignment has been completed will be highlighted in Green. Dates that the daily assignment is only partially completed, will be highlighted in yellow. Dates when no procedures or at least one procedure has not been completed to the assigned time, will be highlighted in white.


2019 - June						
Sun	Mon	Tue	Wed	Thu	Fri	Sat
26	27	28	29	30	31	1
2	3	4	5	6	7	8
9	10	11	12	13	14	15
16	17	18	19	20	21	22
23	24	25	26	27	28	29
30	1	2	3	4	5	6

## Viewing Performance


Each procedure is displayed with a white card that shows Stars earned and procedure status. Click on this white card to view the patient's scores, which are displayed as a line graph and data table.

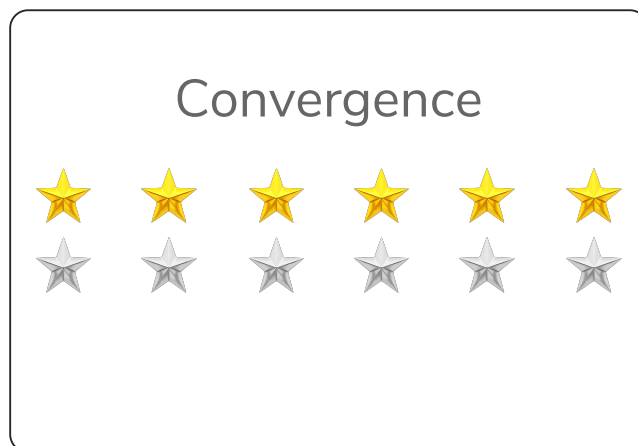


Once all Stars for a procedure have been earned, a Procedure Completed message will be displayed below the Stars and that procedure will no longer appear in the Daily Exercise Regimen.

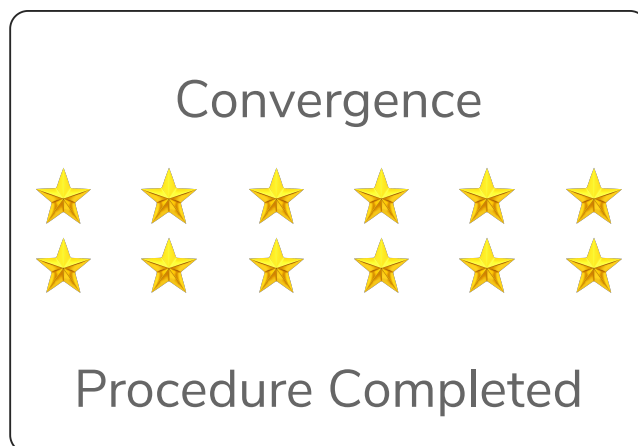

A procedure in Maintenance mode will display In Maintenance. The procedure will continue in the Daily Assignment until the doctor removes it.

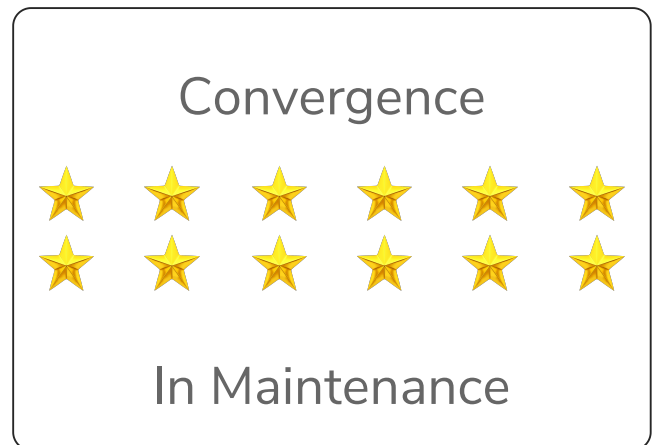

Unassigned procedures will be labeled Not Assigned.

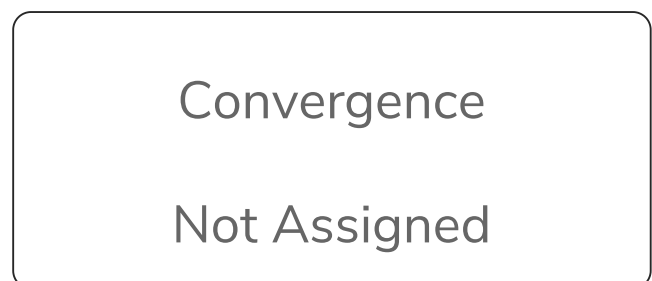




## Procedure Performance Page

To view scores


Click on any assigned procedure.

Scores for the Procedure will be displayed in the line graph.

### Convergence

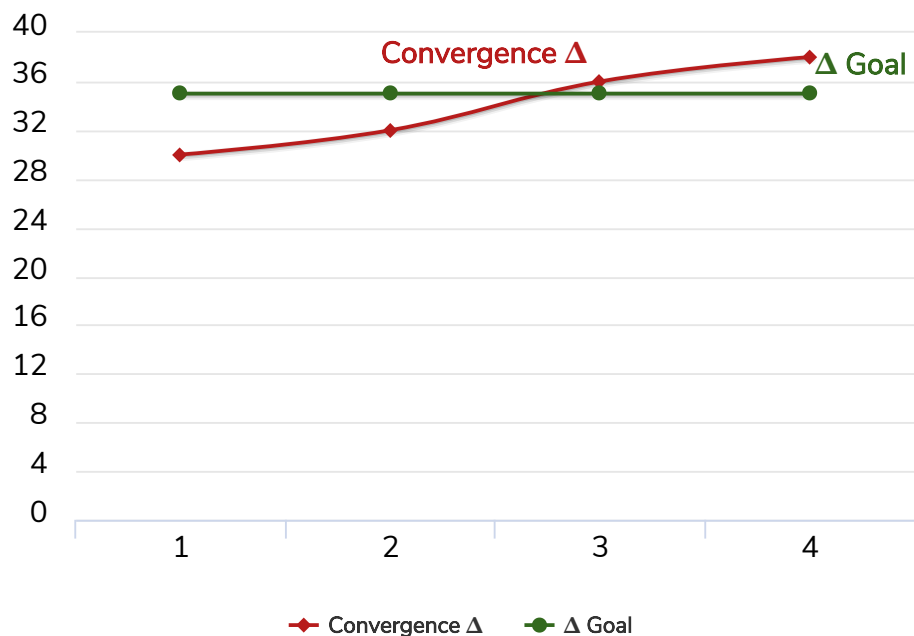

The Stars earned for meeting criteria for a procedure are displayed. Gold Stars are earned while grey Stars are not yet earned.

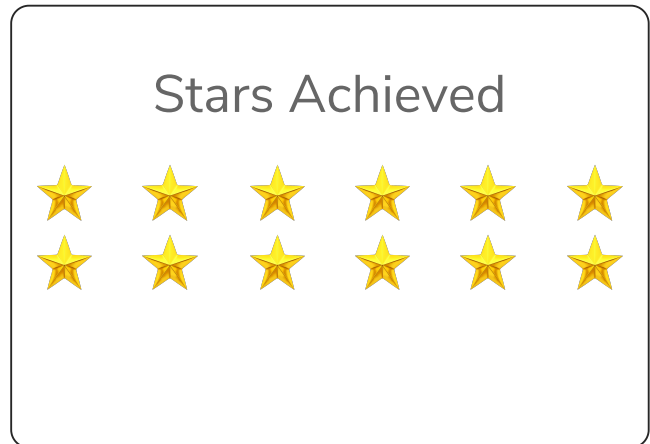

Click Show Scores to see the numeric data.



Each data point and settings for a given procedure are displayed and separated by columns. The MODE column will display the mode the Procedure was ran under and any options that are selectable by the patient. Additionally, the Duration will be displayed in red ( **01:23** ) if the patient exited the procedure before the full Assigned Duration.

<b>Auto</b>	Procedure ran in the Auto Mode.
<b>Maintenance</b>	Procedure ran in the Maintenance Mode.
<b>S</b>	Vergence Procedure Spaceship option.
<b>C</b>	Vergence Procedure Clicker option.
<b>FF/S</b>	Vergence Procedure Flat Fusion Stereo setting.
<b>FF/NS</b>	Vergence Procedure Flat Fusion Non-Stereo setting.
<b>D</b>	Procedure ran with Distractors enabled.

The default date range reflects the last 30 days. To change this date range, select Set Date Range and select your desired dates. Scroll up and down with your mouse wheel, up/down arrow keys, or scroll up and down with your finger on a tablet to move the calendar dates.

Set Date Range

Click Reset Date Range to return to the current 30 days range.

Reset Date Range

Click the blue Print icon in the upper right to print the scores. The printout will include the line graph and the numeric table of scores.



# Contacting Your Patients

## To send a message

Click on the patient's name from the Active Patient List.

First Name	Last Name	Last Use	Messages
Demo	Patient	1 Days since Last Use	
Ben	Wilson	5 Days since Last Use	1
Emma	Smith	1 Days since Last Use	3
James	Brown	2 Days since Last Use	
Sophia	Garcia	1 Days since Last Use	

Click on the message box and type a message.

Send message to John Smith

Click here to begin typing ...

Send Message

Click "Send Message" when complete.

Send Message


On the confirmation prompt, Click "Send" to send the message. You may Click "Do Not Send" if you wish to cancel.



A black notification box will appear at the top that verifies the message was successfully sent.




Your patient will see a red dot in the Contact Doctor button alerting them that a message has been received. The red dot will disappear once the message has been read.

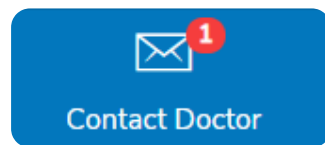


## To View Messages


If you have no new messages, you will see a "No New Messages Available" message in the messages box.

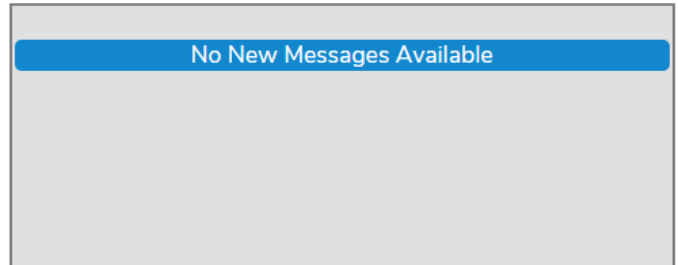

Unread messages are light blue. Click on that message to open it.

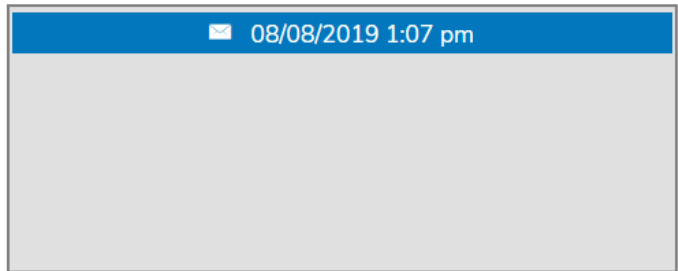

Read messages are dark blue.

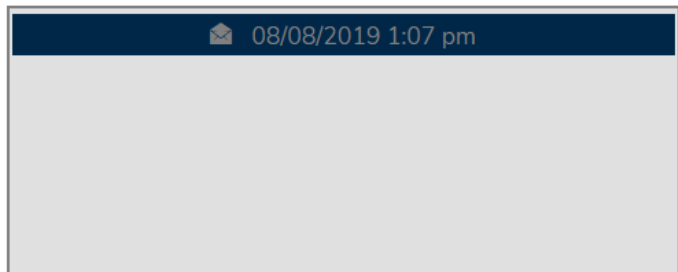

Your patient's message will appear in the center of the mail window.



---

Sent: 08/08/2019 1:07 pm

---

**Doctor Smith,**

**My scores are improving. Are you satisfied with my progress?**

Click the blue "Close" to close the message.  
The message will be marked as Read and may also be viewed again later.




## To Record Notes for a Patient

Click on the Patient Notes message box and type your message.

Record Notes

Click here to begin typing ...



Save Note

When satisfied with your notes, Click "Save Note". Your notes will remain until removed or replaced.






## Customize New Patient's Program

If your patients order CVS2 directly from HTS and you wish to customize the default Daily Exercise Regimen, simply note the protocol you desire in the "Doctor's Modification Request" box on the Patient's Order Form and HTS will make the changes before shipping.

If you are dispensing CVS2 directly from your office and wish to customize the default Daily Exercise Regimen before dispensing Click the Customize New Patient button from the blue Navigation Bar at the bottom of the Active Patients List.

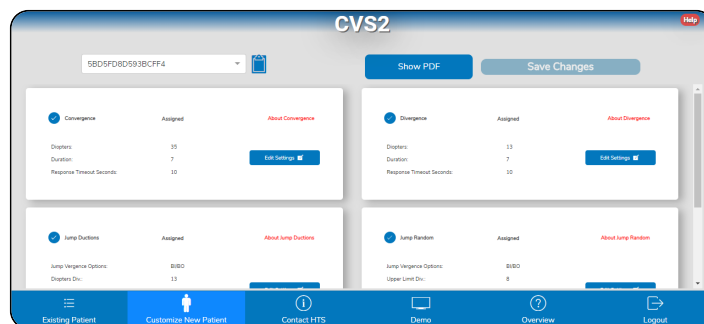

Click the red **HELP** button in the upper right of this page to view detailed instructions on procedure settings and modifications.

## Modify New Patients Procedures

If the CVS2 program has not been dispensed and you wish to modify the default Exercise Regimen, you may do so on this page.

### To select an CVS2 kit to edit

Each CVS2 Kit will contain a Registration Card. The card will contain the Registration Code for this kit.

**CVS2 Registration Card**

**Device Compatibility**

CVS2 Online Computerized Vergence System is designed to work on Chrome, Safari, or Silk Browsers.

Internet Explorer and Microsoft Edge are not supported but Google Chrome is a free alternative. Please call HTS or email us if you wish to install Google Chrome and need assistance.

CVS2 Online Computerized Vergence System will work with most Desktops, Laptops, and Tablets (including iPads, Androids, Chromebooks).

CVS2 will not work on cell phones or devices with a screen smaller than 7.5 inches in width.

**Instructions for registering your CVS2**  
To get started, open your browser and go to: [HTS.Vision](https://HTS.Vision)

At the Home Page, Click/Touch Continue.

On the Welcome Page, Click/Touch the Register button.

After reading the End User License Agreement, Click/Touch on Accept at the bottom of the page to proceed.


You will be prompted for a registration code which is:

After entering the registration code, you will be prompted to create a permanent Username and Password for login.


The Username and Password you create will be used to login in the future.

Be sure to make a record of your Username and Password on the reverse side of this card before clicking Continue.

---

 Tel: 800-346-4925  
Intl Tel: 480-983-0857  
Email: [CVS2@homevisiontherapy.com](mailto:CVS2@homevisiontherapy.com)

Locate this code from the drop down box.  
Click on this code to select it.

## To modify the default Daily Exercise Regimen or Procedure settings before dispensing

A procedure and its settings are displayed as a white card.

The screenshot shows a white card with a blue checkmark icon on the left. The card title is 'Convergence' followed by 'Assigned' and a red link 'About Convergence'. Below the title, it says 'Procedure Completed Maintenance'. The card contains a table of settings: 'Diopters: 35', 'Duration: 7', and 'Response Timeout: 10'. On the right side of the card, there are two blue buttons: 'Reset Stars' with a star icon and 'Edit Settings' with a pencil icon.

For more details on a procedure, click on **About ...** link to see the operating instructions for the procedure.

## About Convergence

A blue checkbox next to a procedure's name denotes if it is assigned to the Daily Exercise Regimen. Click the checkbox to add or remove a procedure from the Protocol.

This screenshot is identical to the one above, but it includes a mouse cursor pointing at the blue checkmark icon in the top-left corner of the procedure card.

To edit any procedure's settings, click Edit Settings.



Click the - or + buttons to change criteria and goal settings.

Click the checkbox to enable or disable additional features.

Click the ? to see additional information about features and settings.

Close X	Convergence	Done ✓
<input type="button" value="-"/> 35 <input type="button" value="+"/>	Diopters	<input type="button" value="?"/>
<input type="button" value="-"/> 8 <input type="button" value="+"/>	Duration	<input type="button" value="?"/>
<input type="button" value="-"/> 10 <input type="button" value="+"/>	Response Timeout	<input type="button" value="?"/>
	<input type="checkbox"/> Maintenance	<input type="button" value="?"/>
	<input type="checkbox"/> Auditory Distractors	<input type="button" value="?"/>

Click Done ✓ when satisfied with your selections.


Done ✓

Then click Save Changes.

Save Changes

## Contacting HTS


Click the Contact HTS button from the blue Navigation bar at the bottom of the Active Patient List page. On this page, HTS current operating hours and contact information is displayed. If you cannot call, you may send HTS an email message from this page.

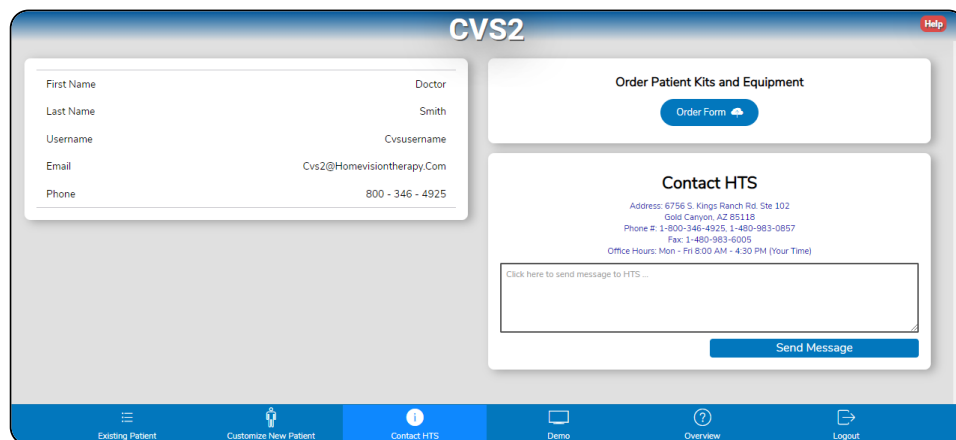

Click the red **HELP** button in the upper right for detailed information.


## Demonstrating CVS2

To Demonstrate/Practice any CVS2 procedures, Click the Demo button from the blue Navigation Bar at the bottom of the Active Patient List page.

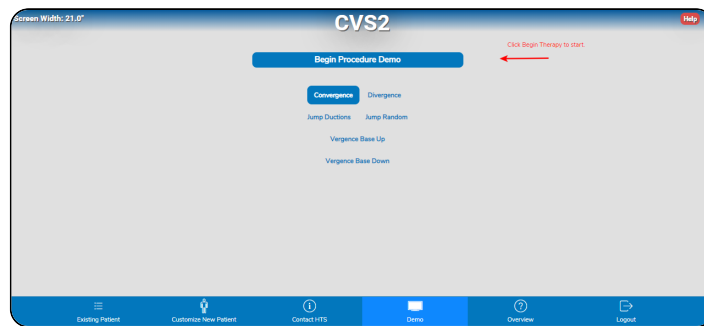

Click the **HELP** button for detailed instructions.

Select any procedure by Clicking on the procedure's name. Click Begin Procedure Demo to start the demo.

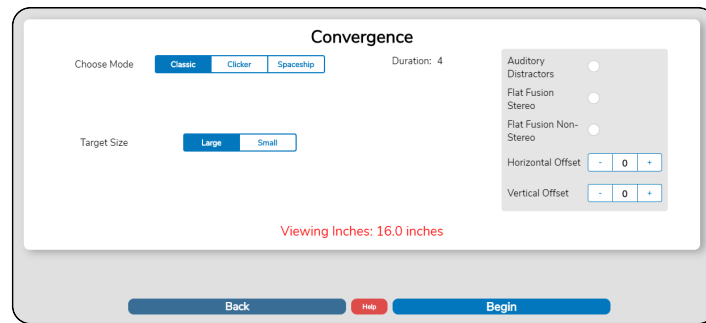

You may Click the red **HELP** button at the bottom of each procedure page for detailed instructions.

The options in the Grey box in the upper right hand corner are options that are deselected in the default Daily Exercise Regimen. You may turn them on or off to demonstrate/practice.

All demo procedures are set to run for 4 minutes. You may press escape if you wish to exit early.

